


User Guide for One-piece Ostomy Bag


1. Clear the skin around the stoma with tepid water or saline.




2. Cut the hole on skin barrier according to actual stoma size and shape.

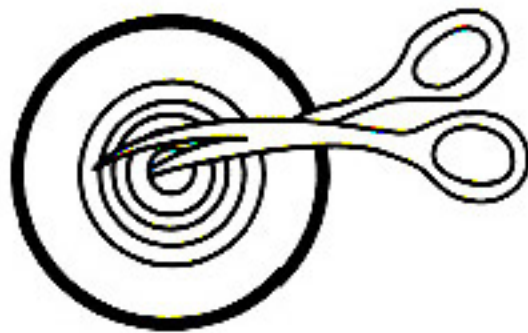

3. Put the gauze on stoma to prevent the exudates from outflow during sticking of skin barrier.

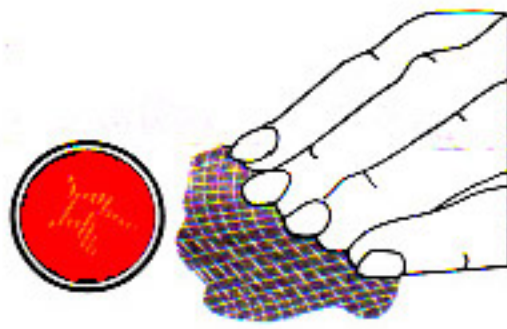


4. Open the clamp.

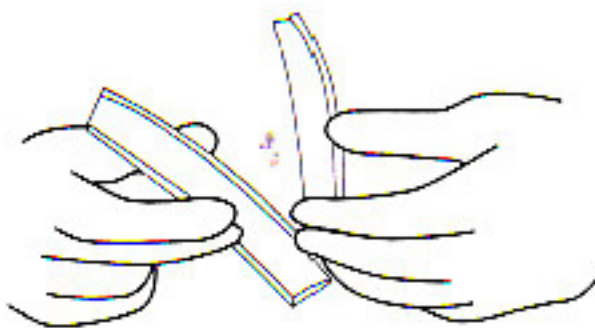

5. Close the ostomy bag with the clamp.



6. Peel off the release paper.

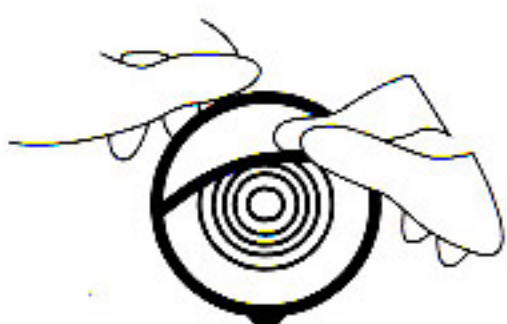

7. Let some air into the bag in case of layers of the ostomy bag sticking together. Put the skin barrier on the stoma area and press it from bottom up to make it completely conglutinate with the skin.



8. Open the end of the ostomy bag and make the excrement out into the toilet. Clean and dry the bag for next use.




9. When taking off the skin barrier, press the skin with one hand and remove the skin barrier by the other hand gently. If it is difficult, put some tepid water on for minutes and then remove the skin barrier.




Caution:

1. It is used for patients with stoma problem.
2. Skin barrier can be used for 5 to 7 days usually, adhesive ability will be affected by continuous use.
3. If the bag is full of air, a small hole can be made on the bag and it is suggested to use Innomed carbon active filter.
4. Ostomy bag can not be recycled.

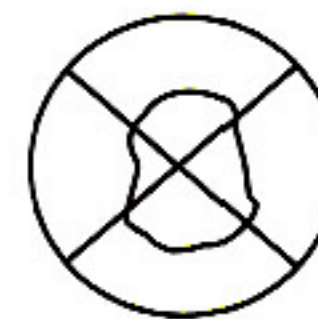

5. Ostomy bag can not be thrown directly into toilet or cloaca, and should not be discarded at will. Please pay attention to the environmental protection.

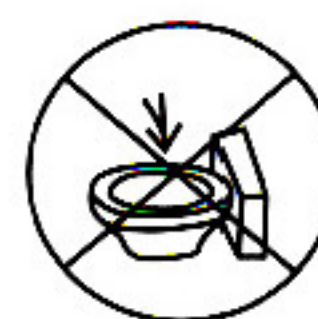

6. Patients allergic to adhesive should use it with caution.