



# User Guide for Two-piece Ostomy Bag


1. Clear the skin around the stoma with tepid water or saline.

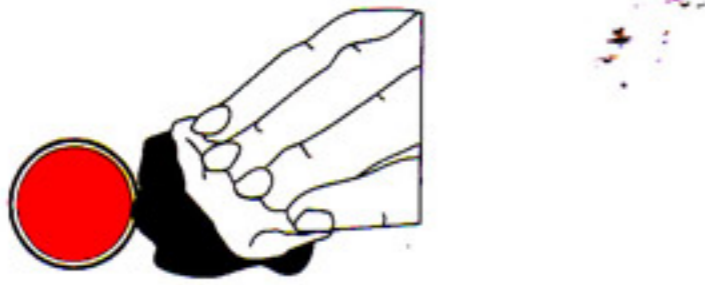


2. Cut the hole on skin barrier according to actual stoma size and shape.

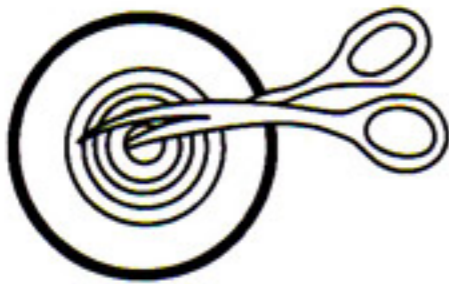

3. Put the gauze on stoma to prevent the exudates from outflow during sticking of skin barrier.

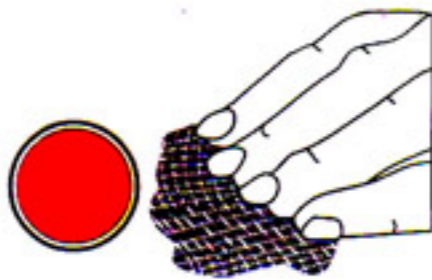

4. Peel off the release paper on skin barrier, put the skin barrier on the stoma area with two hands, press the skin barrier from bottom up to make it stick on the skin flatly.

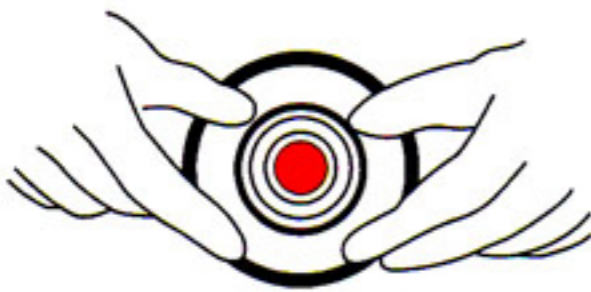

5. Put the plastic ring on the skin barrier buckle ring, and press the toothing with fingers from bottom up. The audible click confirms the attachment of skin barrier to pouch. Pull the pouch to check whether it is safe.



6. Remove the pouch by nipping the tab on the ring.



7. When taking off the skin barrier, press the skin with one hand and remove the skin barrier by the other hand gently. If it is difficult, put some tepid water on for minutes and then remove the skin barrier.



## Caution:

1. Before sticking hydrocolloid skin barrier, make sure that the skin around the stoma is dry and clean. It is better to make the skin around the stoma smooth before using the ostomy bag and press the skin barrier for minutes to enhance the adhesive ability.
2. Please shave the hair if there is too much hair around stoma, or the ostomy bag can not stick well on the skin and folliculitis will occur.
3. Please only clean the stoma from inside to outside with lukewarm water, do not use disinfectant, soap or greasy detergent.
4. The hole of skin barrier should be 1-1.5mm bigger than the stoma, otherwise bleeding of mucous membrane or skin maceration will be caused.
5. If there is swelling for the ostomy bag, there is too much intestinal gas inside the bag, please make some small pinholes on the bag and add active carbon filter.
6. The ostomy bag should be stored in dry room with temperature between 15-25°C, it should not be stored in more than 40°C environment, moist area or refrigerator.
7. Please put the ostomy bag into clean bag and throw it into dustbin. Please do not throw it into the toilet directly.